

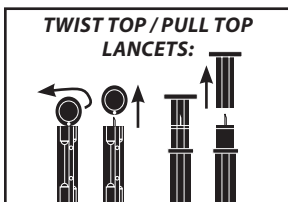
# LANCING DEVICE INSTRUCTIONS FOR USE

For obtaining capillary whole blood for testing.

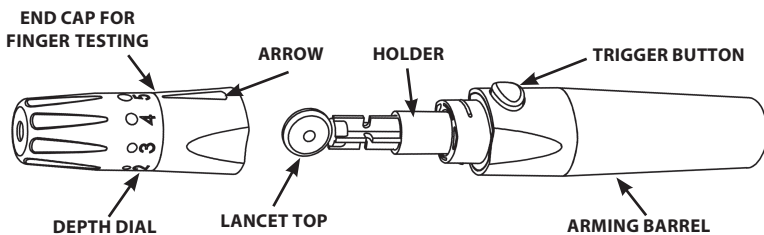CLEAR END CAP FOR FOREARM TESTING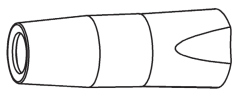


**WARNING!** The Lancing Device is for single patient use **ONLY**. **DO NOT** share your Lancing Device with others, including family members. **DO NOT** use on multiple patients. **ALL** parts of the Lancing Device are considered biohazardous and can potentially transmit infectious diseases, even after you have performed cleaning and disinfection.

**CAUTION!** Do not use lancet if cap has been previously removed. We recommend that you wash your hands thoroughly with soap and water after handling the Meter, Lancing Device, or Test Strips to prevent the transfer of blood-borne viruses or infectious agents.

## - R E A D Y -

1 Select area to be lanced. Wash with soap and warm water, rinse and dry thoroughly.

2 Pull off End Cap.  
Push Lancet firmly into Holder.

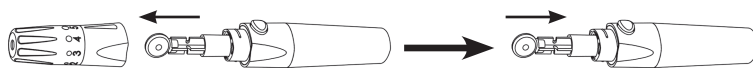3 Remove Lancet Top.  
**Note: Save Top for safe disposal of used Lancet.**

4 Replace End Cap.  
**Note: Use clear End Cap for lancing forearm.**


## - S E T -

### For Finger Testing:

1 Turn Depth Dial to desired lancing depth.  
**Note: Finer adjustments can be made by setting Arrow between numbers.**


Lancing Depth Guide:

- 1-2 for soft / thin skin
- 3-4 for average skin
- 4-5 for thick / calloused skin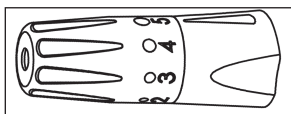


2 Lower hand to waist level. Gently massage finger to get blood flowing.

3 Hold End Cap in one hand and Arming Barrel in other. Gently pull Arming Barrel until a click is heard.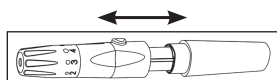


### For Forearm Testing:

1 Use Clear End Cap in place of regular Depth Dial End Cap, but if Clear End Cap is not available, select #5 on Depth Dial.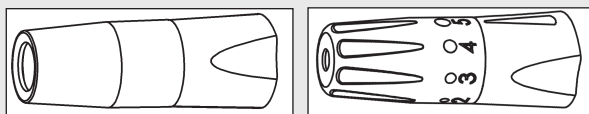


2 Rub area vigorously or apply warm compress to stimulate blood flow.

3 Hold End Cap in one hand and Arming Barrel in other. Gently pull Arming Barrel until a click is heard.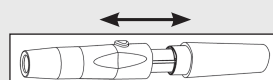

## - L A N C E -


1 Place Tip firmly against finger on other hand.  
**Note: For best results, lance in shaded areas.**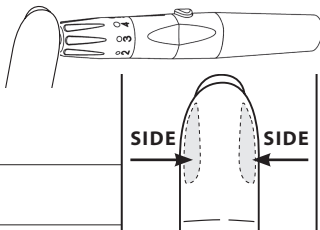

2 Press Trigger Button.

3 Gently massage finger from palm to fingertip to form blood drop.

1 Place Tip firmly against forearm.  
**Note: For best results, lance in shaded area.**

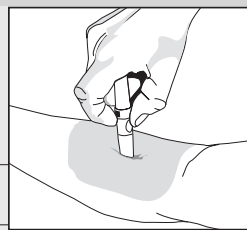2 Press Trigger Button.

3 Apply a firm pressure on the Lancing Device until an adequate blood drop is formed.

### Removing Used Lancet:

1. Pull off End Cap.
2. Using saved Top, recap Lancet.
3. Remove from Holder and dispose of Lancet in appropriate container.

### Cleaning/Disinfecting Lancing Device:

For safe care, clean/disinfect your Lancing Device after each use to prevent the carry-over of blood-borne viruses or infectious agents through your lancing device.

1. Remove End Cap.
2. Wipe the inside of Cap using a cotton swab dampened with 70% isopropyl alcohol.
3. Replace End Cap.
4. Wipe the outside of the Lancing Device with prepared Isopropyl alcohol wipes (active ingredient- 70% Isopropyl alcohol/Isopropanol), PDI Super Sani-Cloth wipes (active ingredients - 55% Isopropyl alcohol/Isopropanol, 5,000 ppm (Parts Per Million) quaternary ammonium chlorides) or any comparable wipes listed on the EPA Disinfectant list ([www.epa.gov/oppad001/list\\_d\\_hepatitisbhiv.pdf](http://www.epa.gov/oppad001/list_d_hepatitisbhiv.pdf)) from [www.epa.gov](http://www.epa.gov).

**Note:** Prepared Isopropyl alcohol (70%) wipes should be available at your local pharmacy. For more information on the PDI Super Sani-Cloth wipes, visit [www.pdipdi.com](http://www.pdipdi.com).

For assistance contact Customer Care at 1-800-803-6025 (USA) or 1-954-677-4599. Available 24/7.

### Important Notes Regarding Forearm Samples Used in Testing Blood Glucose:

- Check with your Doctor or Diabetes Healthcare Professional to see if forearm testing is right for you.
- Blood glucose results from forearm are not always the same as results from fingertip. Do not use forearm for testing when blood sugar is rapidly rising or falling, e.g., within 2 hours of eating, after taking insulin, immediately after exercise, or when you are ill or under stress. Use fingertip sample for testing.
- Do not use forearm samples for testing if you think your blood sugar is low, if you don't regularly experience symptoms when your blood glucose is low, or if your results from forearm samples do not match the way you feel. Use fingertip sample for testing.

**Note: Lancing Device and Clear End Cap are for self-testing and intended for use by one patient only. Not suitable for use by healthcare or care workers.**

- Refer to Owner's Booklet for testing instructions. •

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