Sidekick® All-In-One Blood Glucose Testing System - Instructions for Use

Importance of Blood Glucose Monitoring

Your Doctor or Diabetes Healthcare Professional decides your blood glucose target range and testing frequency. Having most results within your target range helps slow or stop diabetic complications. Do not change your treatment plan before discussing with your Doctor or Diabetes Healthcare Professional.

- Important Information
 Sidekick® All-In-One Blood Glucose Testing System is an *in vitro* (outside body) quantitative system that is for self-testing and point-ofcare testing of human whole blood glucose.
- Read all instructions before testing.
- Write date first opened on vial and Control Solution bottle labels. To assure accurate results:
- Use Controls or Strips before date printed next to EXP.
- Use Controls within 3 months after opening.
- Use Strips within 120 days after opening.
 When all Strips are used, or Sidekick* All-In-One Blood Glucose Testing System has reached expiration date, make sure all results are recorded on log sheet, then discard entire System. **Test Strips cannot be purchased**
- Sidekick® All-In-One Blood Glucose Testing System Test Strips in vial only work with Meter mounted on vial lid. There is no need to code
- Discard vials or bottles that appear cracked or leaking.
 Use only fresh capillary whole blood samples from finger or forearm for


testing. Use of blood from veins or arteries with Sidekick® All-In-One Blood Glucose Testing System is not recommended.

- Test results are displayed as Plasma values.
- Sidekick® All-In-One Blood Glucose Testing System is not intended to diagnose diabetes or for testing newborn blood glucose.
- Use of Sidekick® All-In-One Blood Glucose Testing System in a manner not specified in these Instructions for Use is not recommended and may affect your ability to determine your blood
- When carrying your Sidekick* All-In-One Blood Glucose Testing System with you, be sure it is not left someplace (e.g. jacket pocket, handbag, car, etc.) where the recommended storage temperatures
- Sidekick® All-In-One Blood Glucose Testing System is to be used ONLY with Sidekick® All-In-One Blood Glucose Testing System Test Strips and TRUEcontrol® Control Solution.

WARNING: Upon opening the carton, examine the product for missing, damaged or broken parts. Ensure the vial cap is securely closed. If the product is damaged or the vial cap is not closed, DO NOT use the test strips for testing; product may give inaccurate results. Contact Trividia Health Customer Care at 1-800-803-6025 for replacement and assistance.

For assistance or more information on Sidekick® All-In-One Blood Glucose Testing System, call: 1-800-803-6025, Monday - Friday, 8AM-8PM EST.

Meter/Vial Meter -Display Button Test Port Lot Vial Cap Lip (Lift Up Expiration to Open) Control Level Ranges

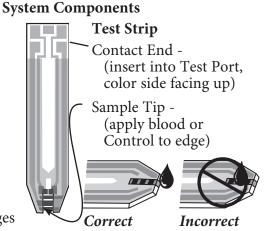Quality Control Testing

To assure that you are getting accurate and reliable results, Sidekick® All-In-One Blood Glucose Testing System offers two types of quality control tests. These tests let you know that your Sidekick® All-In-One Blood Glucose Testing System is working properly so you can have confidence in your test results.

• All meter brands perform differently. Test results from one meter brand to another may vary. This is why test results from your meter should only be compared to a lab instrument and not to another meter brand. To assure your system is working properly, manufacturers of all meter brands recommend performing quality control tests.

An automatic self-test is performed each time a Test Strip is inserted correctly into Test Port. If drop symbol appears in Display after inserting Test Strip, meter is working properly. If "E-4" appears in Display after inserting Test Strip, Meter will not perform test. Call Customer Care for assistance. Control tests using TRUEcontrol® Control Solutions let you know Test Strips are accurate. Control tests should be performed when Sidekick[®] All-In-One Blood Glucose Testing System is first opened; when results do not match how you feel; or if Sidekick® All-In-One Blood Glucose Testing System may have been damaged.

- Control Solutions are for *in vitro* (external) use only. **Do not** inject or drink Controls.
- Do not use Control Level Ranges printed on the Sidekick® All-In-One Blood Glucose Testing System label for your target blood glucose range. These ranges are for Control test results only.

Aug. 28, 2019

EA6942A

31DEC19

56 to 85

Your Sidekick® All-In-One Blood Glucose Testing System includes a Level 1 Control Solution. A Level 0 Control Solution is also available. If you need a Level 0 Control Solution, please call 1-800-803-6025.

Control Testing

1. Allow Sidekick® All-In-One Blood Glucose Testing System and Control Solutions to adjust to a temperature between 68°F-77°F (20°C-25°C) for at least 30 minutes. (If opening for the first time, write date on labels.)

Note: Only use TRUEcontrol® Control Solution provided with System. Using Controls from other systems will give inaccurate results.

2. Check dates on labels.

150 to 210 Caution! Using Controls 3 months past written opened date or past date printed next to EXP may give inaccurate results. Discard out-of-date product. Test with new product.

- 3. Gently swirl contents of Control bottle. Do not shake.
- 4. Remove cap from Control bottle. Squeeze one drop of Control onto clean tissue. Wipe tip of Control bottle with tissue.

immediately by firmly pressing down on Vial Cap Lip.

5. Gently squeeze drop of Control onto piece of unused aluminum

foil or plastic wrap. Wipe tip of Control bottle with clean

tissue. Tightly recap bottle. 6. Open vial by lifting up Vial Cap Lip. Remove one Strip. Close vial

7. Insert Contact End of Strip

into Test Port. Display turns on. Drop symbol appears. 8. Press and release Button. Display shows "C".

9. While holding vial, touch Sample Tip of Strip to drop of Control. Do not remove Strip from Control drop until Display begins countdown.

Note: Do not apply drop to top of Strip.

If Display does not start countdown soon after touching Strip to drop, discard Strip. Repeat test with new Strip and new drop. If condition persists, see Troubleshooting.

Note: Use Strip quickly after removal from vial. Strips out of vial

too long give error message E-3. See Troubleshooting section.

10. Compare result with Control Level Range printed on vial label. 11. Remove Strip and discard.

WARNING! Do not use Sidekick® All-In-One Blood Glucose Testing System for blood testing if Control test results are not within Control Level Ranges. Call Customer Care for assistance.

Blood Testing

1. Allow Sidekick® All-In-One Blood Glucose Testing System to adjust to room temperature for 30 minutes. If opening system for first time, write date on vial label.

Aug. 28, 2019 LOT EA6942A 31DEC19

to Open

Press Down

to Close

2. Check dates on label.

Caution! Using Test Strips 120 days past written opened date or past Expiration Date may give inaccurate results. Discard out-of-date product. Test with new product.

3. Wash hands (and forearm for alternate site testing). Rinse well, dry thoroughly.

4. Open vial by lifting up Vial Cap Lip. Remove one Strip. Close vial immediately by firmly pressing down on Vial Cap Lip.

Note: Use Strip quickly after removal from vial. Strips out of vial too long give error message E-3. See Troubleshooting section.

• **NEVER** reuse Test Strips. **NEVER** wipe Test Strips with water, alcohol or any cleaner. **DO NOT** attempt to remove blood or control sample from Test Strips or clean Test Strips and re-use. Reuse of Test Strips will cause inaccurate results.

NEVER add a second drop of sample to Strip. Adding more samples gives an error message.

- 5. Insert Contact End of Strip into Test Port. Display turns on. Drop symbol appears.
- 6. Lance fingertip or forearm. Allow drop to form.
- 7. With Test Strip still in Meter, touch edge of Sample tip to blood drop and allow blood to be drawn into strip. Remove Test Strip sample tip from sample drop immediately after dashes appear across the meter display.

CAUTION! Holding Test Strip Sample Tip to the blood sample too long after the Meter begins testing may cause inaccurate results.

Note: Do not apply drop to top of Strip.

If Display does not start countdown soon after touching Strip to drop, discard Strip. Repeat test with new Strip and new drop. If condition persists, see Troubleshooting.

- 8. Record result on log sheet.
- 9. Remove Strip and discard.

Note: Removing Strip before result displays cancels test. A result will not be displayed or stored in Memory.

Warning Messages (WARNING!):

- ⇒ Lo: Result is less than 20 mg/dL
- ⇒ HI: Result is greater than 600 mg/dL

Caution! Always repeat test to confirm "Lo" and "HI" results. If results still display "Lo" or "HI", call your Doctor or Diabetes Healthcare Professional immediately!

Important Notes Regarding Forearm Testing¹

- Check with your Doctor or Diabetes Healthcare Professional to see if forearm testing is right for you.
- Results from forearm are not always the same as results from fingertip.
- Use fingertip instead of forearm for more accurate results:
- Within 2 hours of eating
- After taking insulin
- Immediately after exercise
- If you are ill or under stress
- If your forearm test results do not match how you feel
- If your blood sugar may be high or low
- If you don't notice symptoms when blood sugar is high or low

References:
1. U.S. Pood and DrugAdministration. Blood Glucose Meters, Getting the Most Out of Your Meter.
[ElectronicVersion]. Retrieved December 22, 2009 from http://www.fda.gov/MedicalDevices/Safety/
AlertsandNotices/TipsandArticlesonDeviceSafety/ucm109371.htm
2. Hackett, J., Aziz, K., Review Criteria Assessment of Portable Blood Glucose Monitoring in Vitro Diagnostic Devices
Using Glucose Oxidase, Dehydrogenase or Hexokinase Methodology; FDA Draft Documnet, p. 6 (1997).

Control Solution Chemical Composition: water, d-glucose, viscosity enhancing agent, inorganic salts, amaranth, preservatives. Result Range: 20 - 600 mg/dL Sample: Minimum 1 microliter (1 µL) fresh capillary whole blood or Control

Test Time: Less than 10 seconds Result: Plasma value Assay Method: Electrochemical Automatic shut-off: After 2 minutes of non-use Weight (with Test Strips): 1.6 oz. (45.4 g) Memory Size: 50 results (blood or Control) Operating Range: Relative Humidity: 10-90% (Non-condensing)

Room Temperature: 34°F-86°F (1°C-30°C) Hematocrit: 30-55% Note: Use within specified operating conditions only.

Memory

Memory holds 50 blood or Control test results.

Press and release Button to view results in Memory. Last test result (blood or Control) is shown first. Press and hold Button to scroll through Memory results.

Blood test results are shown as "R 1" followed by test result, "R 2" followed by test result, etc.

Control test results are shown as "C 1" followed by test result, "C 2" followed by test result, etc.

Health Related Information

Severe dehydration and/or excessive water loss may cause false low results.2 If you believe you have severe dehydration, contact your Doctor or Diabetes Healthcare Professional immediately.

• Low blood glucose (hypoglycemia) symptoms may include trembling, sweating, intense hunger,

nervousness, weakness, and trouble speaking.

• High blood glucose (hyperglycemia) symptoms may include intense thirst, a need to urinate often, a dry mouth, vomiting, or headache. If you have any of these symptoms, check your blood glucose. If your result does not match the

way you feel, repeat the test. If your results still do not match the way you feel, call your Doctor or Healthcare Professional

Care and Storage

WARNING! DO NOT share your Meter with anyone, including family members. ALL parts of your Blood Glucose Testing System could carry blood-borne diseases after use, even after cleaning and disinfection. Cleaning removes blood and soil, disinfecting removes infectious agents.

• Store Sidekick® All-In-One Blood Glucose Testing System and Control in a dry place between 36°F-86°F/2°C-30°C. DO NOT REFRIGERATE OR FREEZE.

When carrying your Sidekick® All-In-One Blood Glucose Testing System with you, be sure it is not left someplace (e.g. jacket pocket, handbag, car, etc.) where the recommended storage temperatures may be exceeded or the System may be crushed.

To Clean the Meter (removing blood or soil):

• For multi-patient use facilities, refer to Customer Care for proper cleaning and disinfection of the system.

Wipe Meter with clean, lint free cloth, dampened with one of the following:

- Mild detergent or mild soap,

- 70% isopropyl alcohol (isopropanol) Let Meter air dry thoroughly before using to test.
 - Do not use bleach to clean the Meter.
- Store Strips in original vial only. Never store Strips outside of vial.
- Do not reuse Strips.

HI

Do not bend, cut or change Strips in any way.

Troubleshooting

- 1. If Display does not turn on after inserting a Strip: Remove Strip. Re-insert Strip fully into Test Port.
- 2. After sample is drawn into the Strip, Display does not begin
- Repeat test with new Strip and a larger sample.
- Repeat test with new Strip. Apply drop until countdown begins.
- 3. If error message appears in Display:

Wait 30 minutes for Sidekick® E-1 ⇒ All-In-One Blood Glucose Testing System to reach room temperature

before testing. Retest with new Strip and larger sample. If you have symptoms such as fatigue, excess urination, thirst or blurry vision, follow your healthcare professional's

advice for high blood glucose. E-3 ⇒ Repeat with new Strip, making sure that Strip has not been used or out of the vial too long.

E-4; Lb; db ⇒

Call Customer Care at 1-800-803-6025.

Testing capacity has been reached. Meter will no longer test.

Note: For any other ERROR messages please call Customer Care.

Manufactured by: Trividia Health, Inc. Fort Lauderdale, FL 33309

© 07/2018 Trividia Health, Inc. Sidekick is a trademark of Trividia Health, Inc. D4TVH04 Rev. 40

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Fabricado por: Trividia Health, Inc. Fort Lauderdale, FL 33309